Saturno 300 Plus

every patient is a stat

CRONY[®]
INSTRUMENTS srl

Saturno 300 Plus

every patient is a stat

SATURNO 300 Plus is a fast random access analyzer for Clinical Chemistry and Immunoassay, where **EVERY PATIENT IS A STAT**

SATURNO 300 Plus is 24 hours on duty to run your daily routine, a single test, a profile, an electrolyte, an organ panel or a STAT with the same speed and efficiency.

State of the art technology with sophisticated user friendly software offers a new dimension in versatility, speed and reliability. It enables each user to personalize the analyzer to fit his particular needs and becomes an indispensable collaborator in any laboratory.

300 tests per hour plus 160 ISE electrolytes at the same time.

A total of 460 tests per hour

Operation Cycle

- Reagent is transferred into a reaction cuvette and brought to temperature prior to the addition of the sample.
- Cuvette with its reagent is immediately measured in the built-in photometer to obtain a Reagent/Cuvette Blank.
- The reaction cuvettes are immersed in a liquid bath at 37°C and are automatically washed for a non stop operation.
- Sample is transferred to ISE module, if required.
- Automatic print-out of patient report.

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
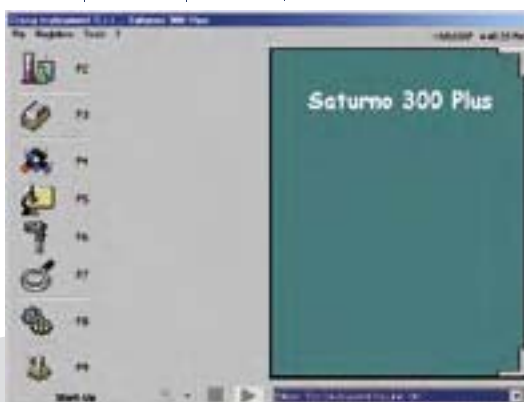
Il Programma

- F2** Will guide the operator step by step through all operations.
- F3** Can store thousands of patient data for future consultation on-line and to print patient reports.
- F4** To introduce an unlimited number of user programmable methods.
- F5** To program an unlimited number of profiles or organ panels.
- F6** To program an unlimited number of standards, calibrators and multi-standard curves.
- F7** To introduce an unlimited number of controls for the Quality Control program.
- F8** On-line control of all operating steps and procedures in the system. On-line trouble shooting.
- F9** Special personalization of ones analyzer.

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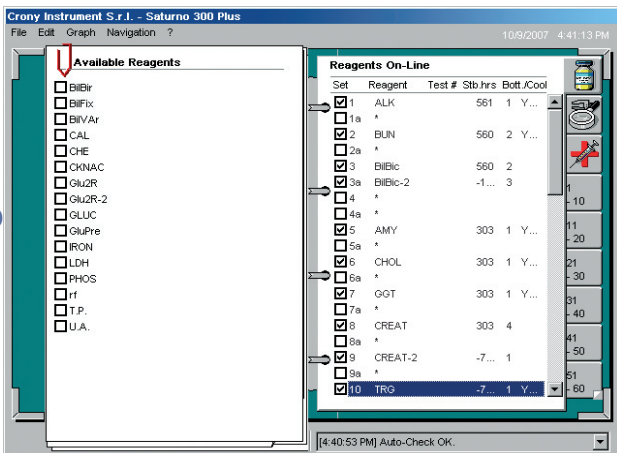
Reagent Control

- 60 On-line reagents individually cooled or left at room temperature.
- On-line reagent inventory and display of the number of available tests in each of the reagent containers.
- On-line reagent stability check.
- Bar code reader for reagent identification (optional).


Patienty entry

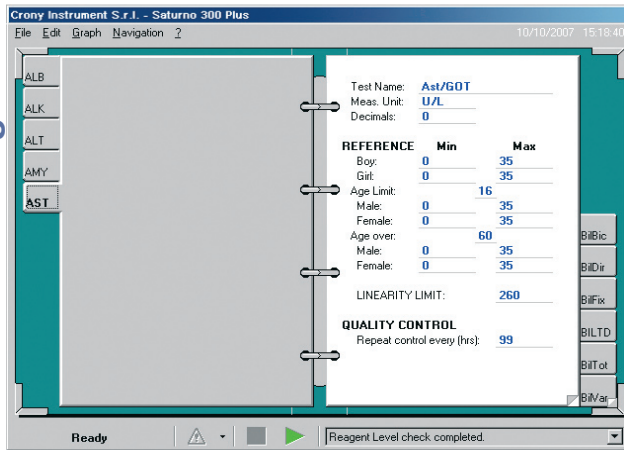- 40 On-line primary tubes or sample cups with continuous sample loading and programming while analyzer is in operation.
- STATS will be processed within 6 minutes, without disrupting routine testing.
- Bidirectional interface enable to receive and transmit data from and to outside computer.
- Bar code reader for sample identification (optional).

Methodologies

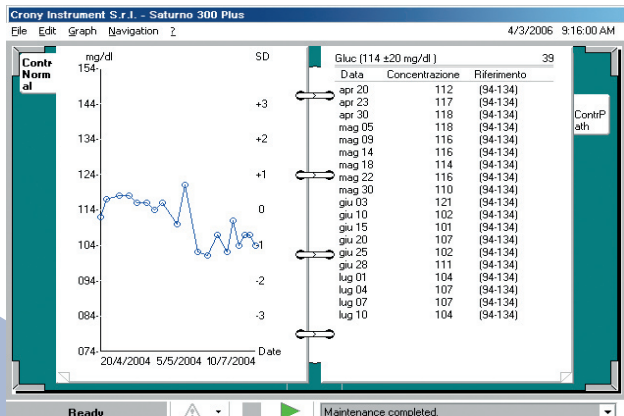- Open system, unlimited number of methods are all user programmable.
- Only 300 µl of reagent per test.
- Automatic sample pre-dilution, if necessary.
- Two reagents chemistries and immunoassays.


Quality Control Graph

A 60 days on-line quality control with its actual values.

Typical Kinetic Measurement

Graph that can be displayed for every sample.

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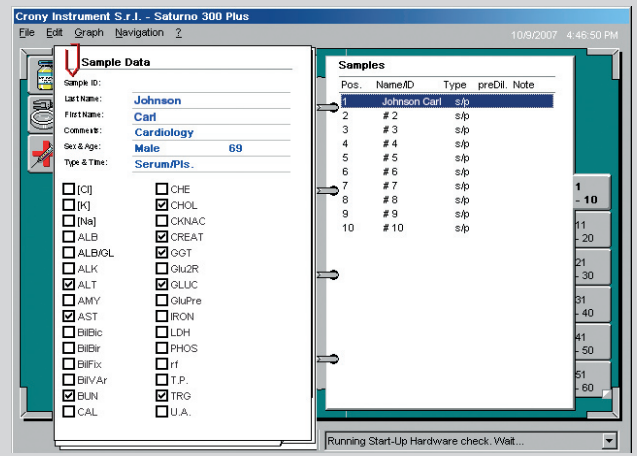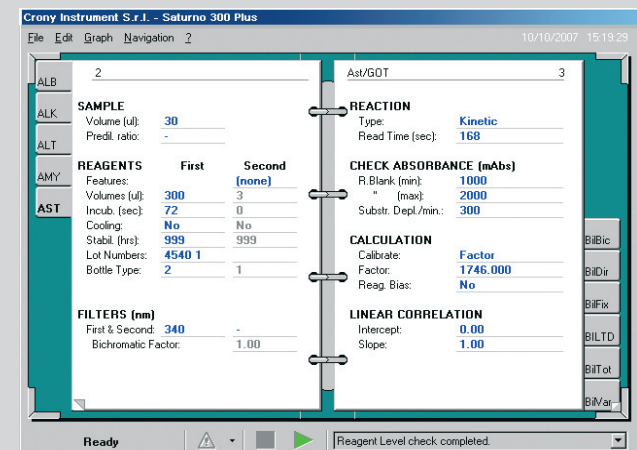
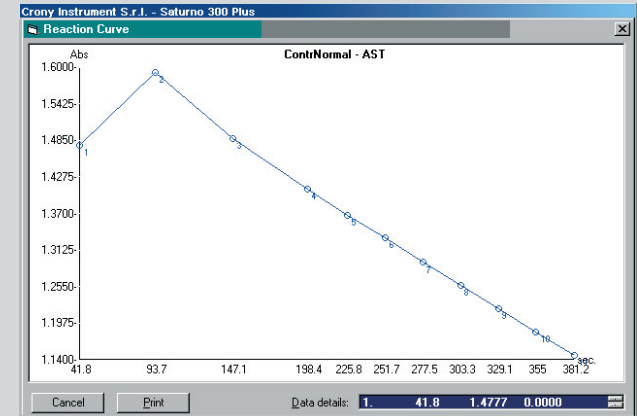
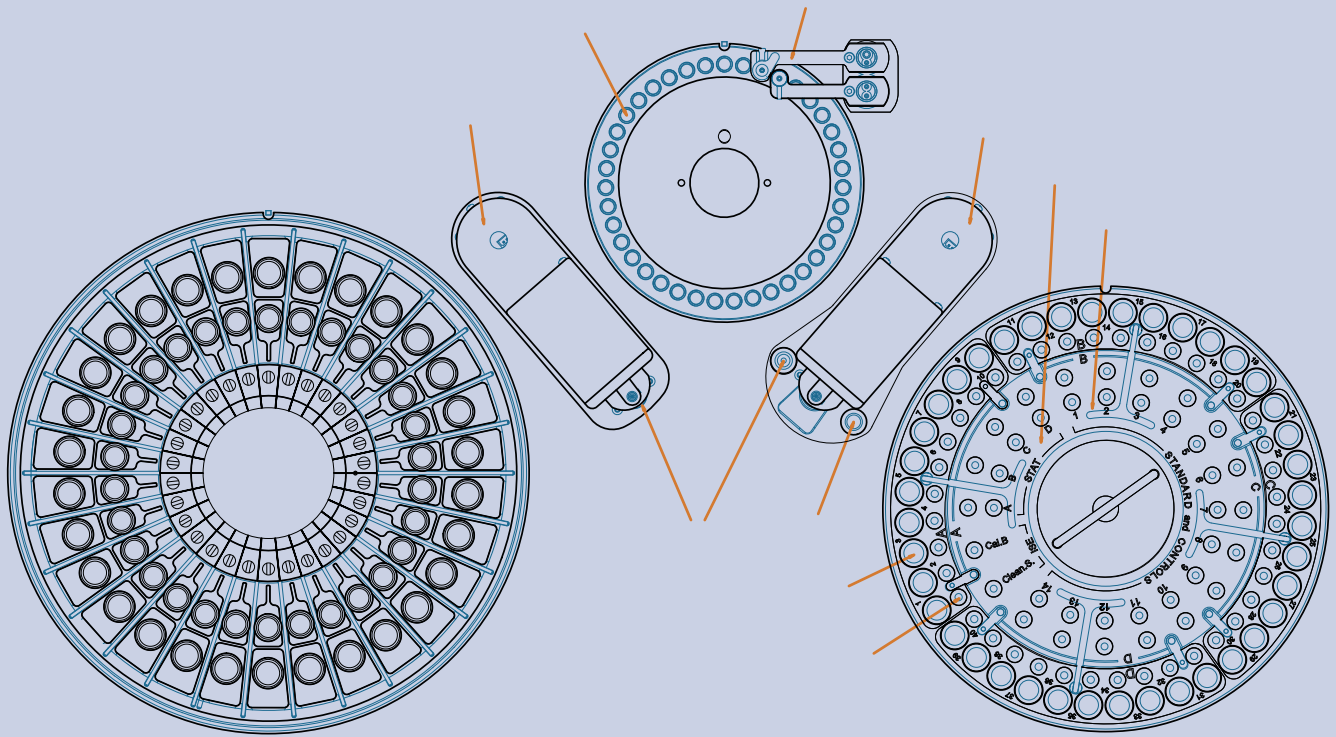
Saturno 300 Plus


Highlights

- Continuous patient programming while analyzer is in operation.
- 60 On-line reagents individually cooled or left at room temperature.
- Cost effective operation - no disposables.
- Walk away system - the reaction cuvettes are automatically washed and dried to assure a non stop, continuous operation.
- Liquid sensor probes for sample preparation and reagent dispensing with built-in mixer.
- Check and control of reagent inventory and stability, flagged when reagent is missing or expired.
- Automatic repetition of critical or out of range results.
- I.S.E. module for simultaneous determination of Na, K and Cl (optional).
- Identification of sample and reagent with a built-in bar code reader (optional).
- Check of the quality of the washing and incubation water.
- Help on-line.

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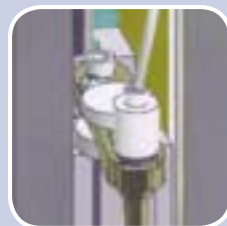OPERATING	True Random Access
ASSAY METHODS	End point, Fixed Time, Kinetic, Bichromatic, Differential, Immunoassay using Multistandard, Log/logit, or Cubic calibration.
60 REAGENTS ON-LINE	Individually cooled or at room temperature.
CHEMISTRY MENU	Open System with unlimited number of methods, profiles, standards, calibrators and controls - user programmable.
TROUGHPUT	300 Chemistry tests plus 160 ISE's/hour ($\pm 5\%$)
RE-RUN OF TESTS	Automatic or upon request
SAMPLE TRAY	40 On-line samples on 4 racks in primary tubes or plastic cups that can be continuously added up to 240 patients at a time. Each patient sample has a corresponding plastic cup for automatic predilution, if necessary. 20 extra places are reserved for standards, calibrators, controls and STATs.
SAMPLE VOLUME	3 μl to 70 μl (with 1 μl increments).
REAGENT VOLUME	R ₁ from 3 to 600 μl (with 1 μl increments). R ₂ from 3 to 597 μl (with 1 μl increments).
REAGENT CONSUMPTION	Only 300 μl per test.
PHOTOMETER	Analytical plate with 39 quartz cuvettes immersed in a 37°C liquid bath. A built-in multi-filter photometer measures the samples at 8 wavelengths continuously during their incubation. A built-in automatic cuvette washing system assures a non-stop operation.
LIGHT SOURCE	Double halogen lamp - 12V / 20 Watt.
DETECTOR	Silicon photodiode.
CUVETTE VOLUME	600 μl .
OPTICAL PATH	7 mm.
PHOTOMETRIC LINEARITY	From -500 to 2.800 Abs $\pm 1\%$.
PHOTOMETRIC SENSITIVITY	0.0005 Abs.

CHARACTERISTICS:

- Industrial Pentium Computer 600 Mhz, 256 Mb RAM, Hard Disk > 80 Gbyte, Floppy disk 1.44 Mb, double serial port.
- Bidirectional interface to EDP system.
- Innovating graphic software in Windows XP Emb. on a large colored screen.
- Liquid sensor probes for sample preparation and reagent dispensing with built-in mixer.
- Check and control of reagent inventory and stability with signalling when reagent is missing or expired.
- Power Supply - 220/115 Vac 50/60 Hz - 500 W.
- Dimensions - 103x67x45 cm.
- Weight - 104 Kg.

Technical Specifications are subject to change without prior notice.

CE Certified: EN 61326-1; EN 61010-1/A1; IEC 61010-2-081; EN ISO 13485:2003 — **IVD**

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